



# → MECTRON PROPHYLAXIS HIGHER PERFORMANCE FOR BRIGHT RESULTS

mectron

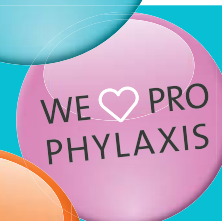


# → ALWAYS THE BEST CHOICE FOR ALL CLINICAL NEEDS

Supragingival biofilm removal

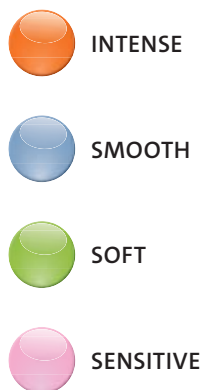

Removal of stains and pigmentations

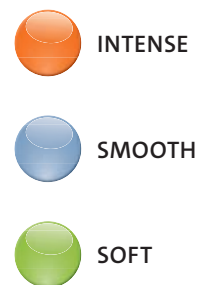


Subgingival biofilm removal




Periodontal pocket detoxification




Implant maintenance

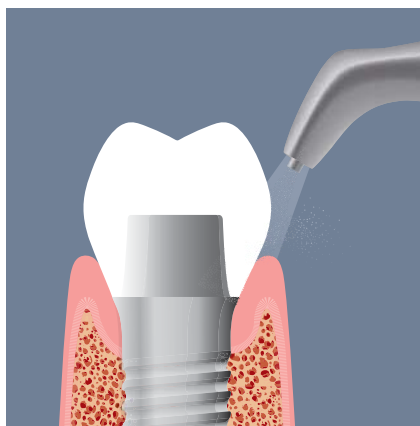

Peri-implantitis treatment

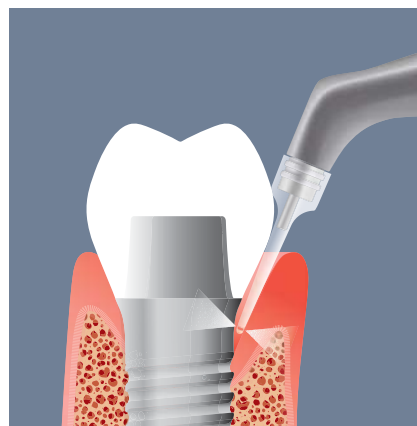

Fixed orthodontic appliances

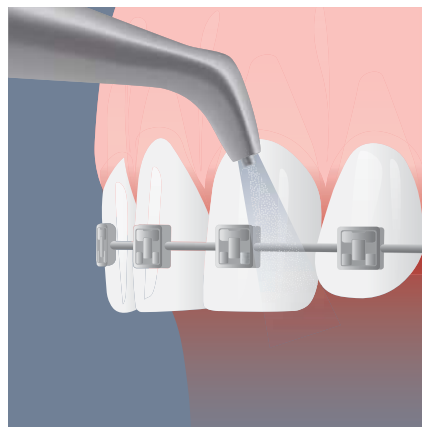

- SMOOTH
- SOFT
- SENSITIVE

Preparation for whitening



- SMOOTH
- SOFT
- SENSITIVE

Cleaning of prosthesis and restoration materials



- SMOOTH
- SOFT
- SENSITIVE

Cleaning of cavity margins before etching



- INTENSE
- SMOOTH
- SOFT

Cleaning of fissures before sealing



- INTENSE
- SMOOTH
- SOFT
- SENSITIVE

Restoration maintenance



- SMOOTH
- SOFT
- SENSITIVE



# → TAILOR-MADE SOLUTIONS TO SUIT THE NEEDS OF YOUR PATIENT

## → PROPHYLAXIS POWDER INTENSE

effectively removes supragingival biofilm and stubborn extrinsic stains on the enamel surface

### INDICATIONS:

- Removal of stains and extrinsic discolorations due to smoke, wine, tea and coloring agents
- Cleaning of fissures and cavities
- Removal of partially mineralized deposits

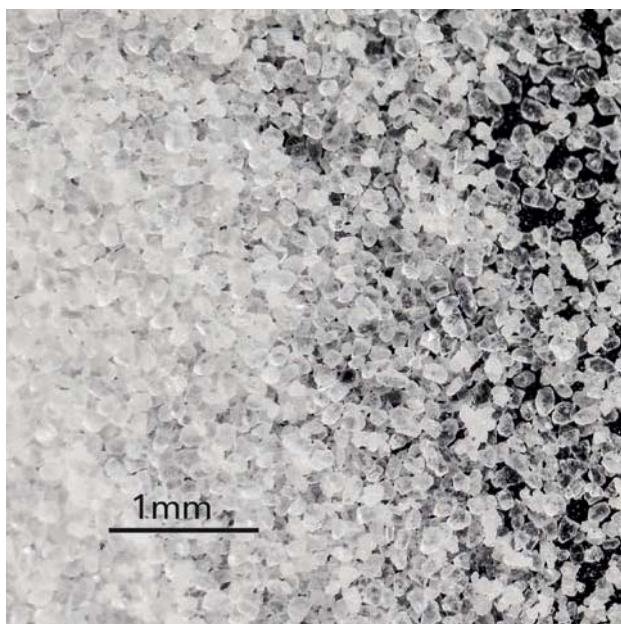


image magnification of 1 mm showing the shape of sodium bicarbonate particles

### PROPHYLAXIS POWDER INTENSE

- Sodium bicarbonate-based
- Average particle size less than 120  $\mu\text{m}$
- Lemon flavor
- Package: 2 bottles of 250 g each
- Ref no. 03140002



## → PROPHYLAXIS POWDER SMOOTH

gently removes, thanks to its spherical shape, supragingival biofilm while respecting the surrounding tissues

### INDICATIONS:

- Teeth cleaning to a shiny finish, while avoiding enamel damage
- Allows better access to the areas difficult to reach
- Aids in reducing the PH acidity in the mouth
- Reduction in aerosol effect and risk of device clogging
- Sodium and silicon (Si) free
- The absence of sodium makes it possible to treat patients maintaining a salt-free diet

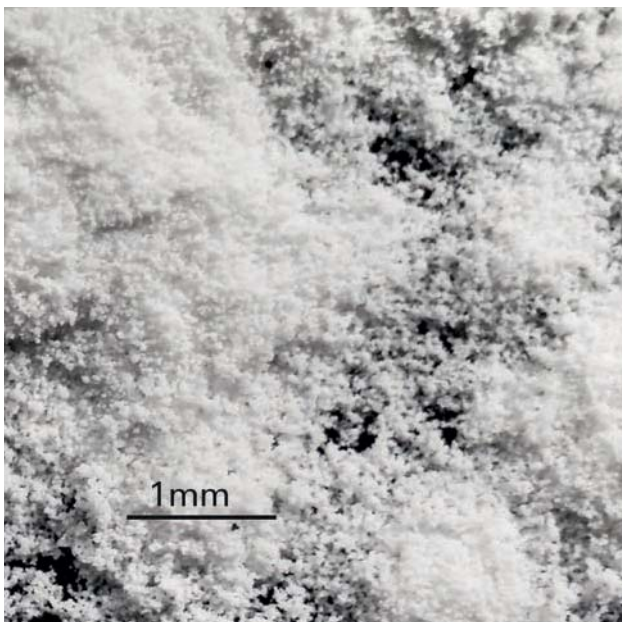


image magnification of 1 mm showing the calcium carbonate particles with spherical geometry

### PROPHYLAXIS POWDER SMOOTH

- Calcium carbonate-based
- Average particle size between 45  $\mu\text{m}$  and 75  $\mu\text{m}$
- Neutral taste
- Package: 4 bottles of 250 g each
- Ref no. 03140010

# → TAILOR-MADE SOLUTIONS TO SUIT THE NEEDS OF YOUR PATIENT



## → PROPHYLAXIS POWDER SOFT

gently and effectively removes supragingival biofilm and stains on the enamel surface

### INDICATIONS:

- Removal of stains and extrinsic discolorations due to smoke, wine, tea and coloring agents
- Cleaning of fissures and cavities
- Cleaning of orthodontic brackets

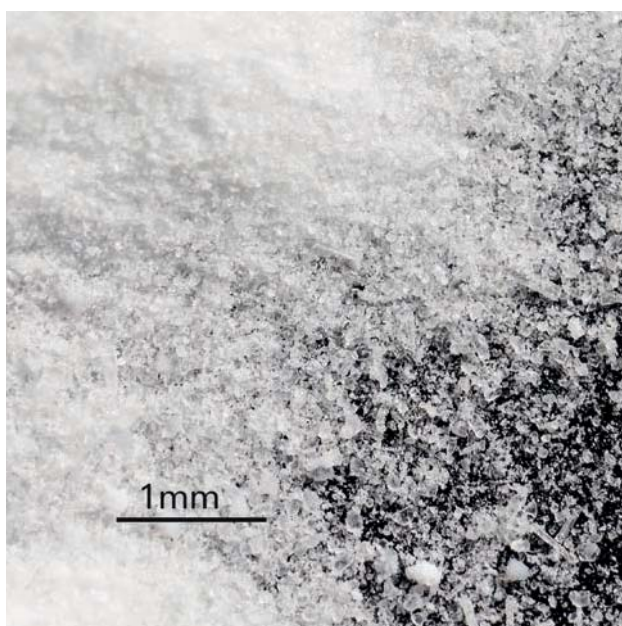


image magnification of 1 mm showing the shape of sodium bicarbonate particles

### PROPHYLAXIS POWDER SOFT

- Sodium bicarbonate-based
- Average particle size 40  $\mu\text{m}$
- Mint flavor
- Package: 4 bottles of 250 g each
- Ref no. 03140008



## → PROPHYLAXIS POWDER SENSITIVE – GLYCINE POWDER


gently removes supra and subgingival biofilm

### INDICATIONS:

- Mild pigmentation and stain removal
- Periodontal pocket detoxification
- Periodontal maintenance
- Implant maintenance and treatment of peri-implantitis
- Orthodontic patient maintenance
- Gentle cleaning of restoration materials and comfortable treatment on exposed root surfaces
- Reduction of bacteria colony-forming units (CFU)
- Can be successfully used for the treatment of each type of patient



Available only in Europe

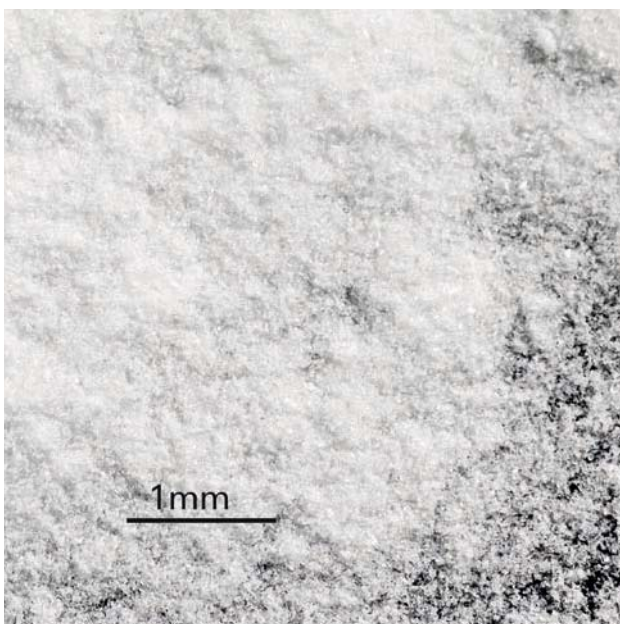

### PROPHYLAXIS POWDER SENSITIVE – GLYCINE POWDER\*

- Glycine-based
- Average particle size 25  $\mu\text{m}$
- Sweet taste
- Package: 2 bottles of 160 g each
- Ref. No 03140007

image magnification of 1 mm showing the small sized and minimally invasive glycine particles



# → THE EVOLUTION OF PROPHYLAXIS



## → STARJET

→ One single air-polisher to manage all Mectron powders with different granulometries to perform a complete supra and subgingival prophylaxis treatment



## → COMBI *touch*

→ air-polishing and ultra-gentle ultrasounds in one unit



For more information about the prophylaxis world by Mectron visit the website

→ [www.we-love-prophylaxis.com](http://www.we-love-prophylaxis.com)